



Quick Fit™ Hinged Knee

**New functional Knee Brace
with customisable patella donut
for targeted support**



Quick Fit™ Hinged Knee

Indications



- Protection, compression and support for knee sprains, mild to moderate ligament lesions, ligament instabilities, meniscus injuries, patella subluxation, laxity or misalignment, mild arthritic pain, return to mobility or sports activities



Targeted Patella Support


- Determine the support required to address patella misalignment:

Superior, Inferior, Lateral or Medial

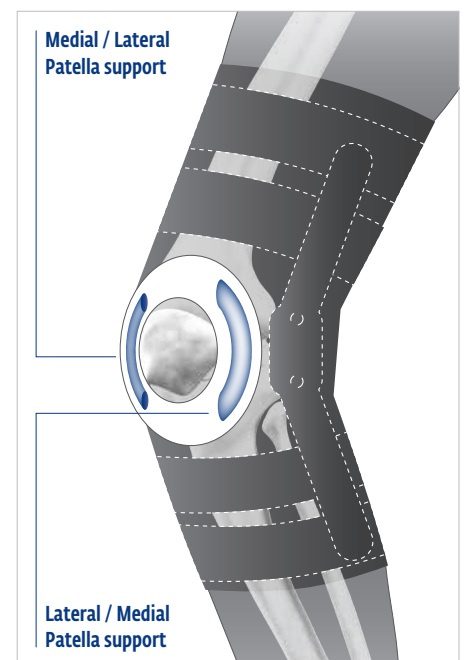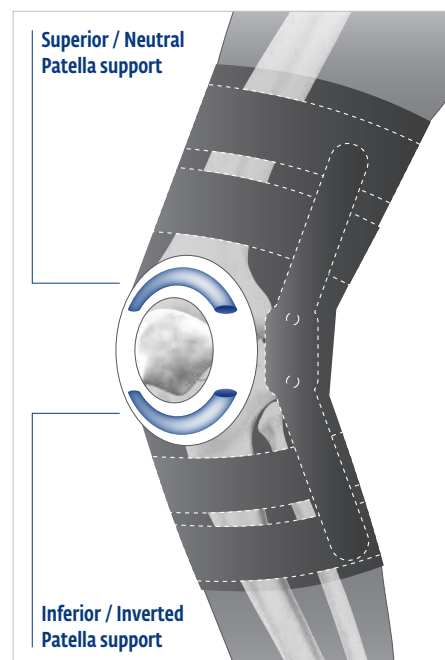


To trim the patella donut


1. Remove the hinges and turn the brace inside out.

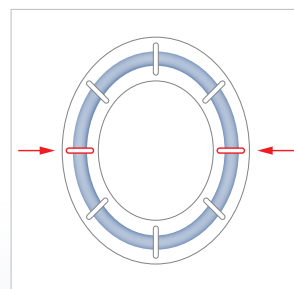

2. Find best pre-cut opening in patella donut cover

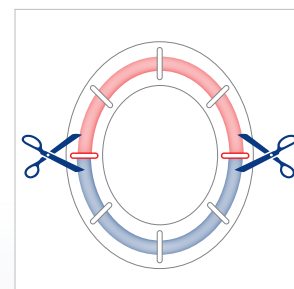

3. Cut the section of the patella donut that needs to be removed with a pair of scissors

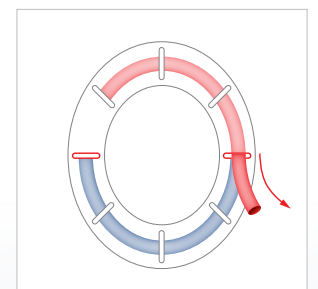

4. Remove the section of the patella donut not required

5. Return brace and hinges to original position

Adjustable Compression

- 6 independent straps (4 anterior and 2 posterior) allow compression levels to be adjusted according to needs
- Straps also enable optimal individual fit

Medial & Lateral Support

- Removable aluminium polycentric hinges offer medial / lateral stability
- Bilateral stays provide support to unstable ligaments

Covered Hinges

- The hinges are encased for added protection from lateral impacts (ideal for sports use)

Tailored Patella Support

- Unique oval-shaped donut helps maintain patella in correct position
- The patella donut can be removed and trimmed to provide tailored correction in cases of patella misalignment

Popliteal Comfort

- Soft, anti-pinch posterior lycra panel enhances comfort
- Allows unhindered movement, even during flexion

Breathable, Form-fitting 5 mm Spacer Mesh Material

- Ideal neoprene alternative, promotes airflow without restricting movement during mobility or sports activities

Easy Fit

- Finger loops for effortless application

Universal


- Fit Left or Right knee
- Available in 7 sizes (XXS to XXXL)





ORDERING INFORMATION

QUICK FIT™ HINGED KNEE			
Part Number	Size	Thigh circumference 15 cm (6") above Mid-Patella	
82-0010-1	XXS	26,5 - 33 cm	10 ½" - 13"
82-0010-2	XS	33 - 39 cm	13" - 15 ½"
82-0010-3	S	39 - 47 cm	15 ½" - 18 ½"
82-0010-4	M	47 - 53 cm	18 ½" - 21"
82-0010-5	L	53 - 60 cm	21" - 23 ½"
82-0010-6	XL	60 - 67 cm	23 ½" - 26 ½"
82-0010-7	XXL	67 - 75 cm	26 ½" - 29 ½"
82-0010-8	XXXL	75 - 81 cm	29 ½" - 32"

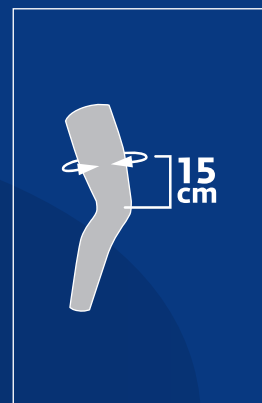

DJO South Africa (Pty) Ltd | Unit 1, Brackengate Business Park | 5 on London | Brackenfell, 7560 | Cape Town | SOUTH AFRICA

Tel: +27 (0) 87 3102480 | **Fax:** +27 (0) 86 6098891

Email: info.southafrica@DJOglobal.com

www.DJOglobal.com



MOTION IS MEDICINE™