ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

IF YOU ARE USING THE CRYO/CUFF WITH THE AIRCAST CRYO/CUFF IC UNIT BE SURE YOU READ AND UNDERSTAND ALL INSTRUCTIONS LOCATED IN THE AIRCAST CRYO/CUFF IC INSTRUCTIONS FOR USE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. User should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

OPERATING PRINCIPLE: The AirCast Cryo/Cuff provides cold therapy and compression.

INTENDED USE/INDICATIONS: The Aircast Cryo/Cuff combines focal compression with cold to minimize swelling and pain.

FREQUENCY OF USE: The length of use and frequency of use of the Cryo/Cuff are determined by the healthcare professional depending on individual patient's needs. This device can be cold enough to cause serious injury so be sure to check skin frequently.

CONTRAINdications: Cryotherapy should not be used on persons with Raynaud's disease, cold hypersensitivity, decreased skin sensitivity, or other vasospastic disease, cold hypersensitivity, decreased skin sensitivity, compromised local circulation, diabetics, sickle-cell anemia, arteriosclerosis, cryoglobulinemia, hypercoagulable clotting disorders or other peripheral vascular diseases causing ischemia or poor local circulation, local tissue infection, cold allergy, cold hypersensitivity, previous cold injury and paroxysmal cold haemoglobinuria.

Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND CAUTIONS:

- This device can be cold enough to cause serious injury, including tissue necrosis. You must be able to check your skin condition under the Cuff (DO NOT use if you cannot check your skin condition frequently). People are sensitive to cold and may experience numbness, tingling, increased redness, decreased coloration, increased swelling, blisters, irritation or other changes in skin condition under the Cuff or around the treatment area. If you experience any of these conditions, immediately discontinue use of the device and contact your physician.
- It is recommended that at least 20 minutes of cessation should be encouraged between 2 hour long cryotherapy sessions.
- Check for redness. If moisture is present on the Cuff, immediately discontinue use of the device.
- Do not use an elastic wrap in conjunction with the Cryo/Cuff.
- Dressings used under the Cryo/Cuff should be applied lightly.
- When filling the Cryo/Cuff do not raise the cooler higher than 15 inches (38 cm) above the cuff to avoid excessive pressure.
- Reduce pressure with any sense of discomfort, numbness or tingling of the limb.
- The Cryo/Cuff is designed for single patient use only and may be used on the same patient for the length of treatment.
- The device should be cleaned if it is used on the same patient for an extended period of time (see "Cleaning Instructions" for cleaning information).
- Keep small parts and packaging materials away from small children and animals. These items pose a risk for suffocation or strangulation.
- It could be unsafe to use accessories, detachable parts and materials, or interconnect to other equipment not described in these instructions, or otherwise modified equipment.

APPLICATION INFORMATION:

1. **PREPARE COOLER -** Connect blue tube to cooler. Add water to line inside cooler, then add ice to top of cooler. Lay the insulation disk on top of ice.

2. **APPLY THE EMPTY CUFF -** Position cuff so that the quick-disconnect is pointing up, so that it conforms to a slightly bent knee or elbow and secure straps (Fig. 1). Tighten straps to a comfortable compression.

3. **FILL CUFF -** Connect blue tube to the cuff, listen for a click/snap. When using the Gravity Cryo/Cuff (non-motorized), open air vent on cooler lid and raise cooler no more than 15 inches (38 cm) above the cuff (Fig. 2). If the cooler is raised more than 38 cm, the cuff will increase in volume and cool faster.


4. **REFILL WATER -** Reconnect the tube to the cooler below the cuff, allow the water to mix with the ice and rechill, then raise the cooler if needed.

5. **CLEANING INSTRUCTIONS:** After use, completely drain water from cuff, tube, and cooler. (To drain tube, elevate tube while pressing tip of tube/disconnect.) Store cooler with top off to allow drying. Periodically clean cuff, tube, and cooler using a few ounces of liquid soap added to hot water in cooler. Recipe soap/water mixture between cooler and cuff a few times, then repeat with warm water only.

6. **MATERIAL CONTENTS:** Polyurethane Nylon, Nylon, Polyether foam.

7. **WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

8. **EXPLANATION OF SYMBOLS:**

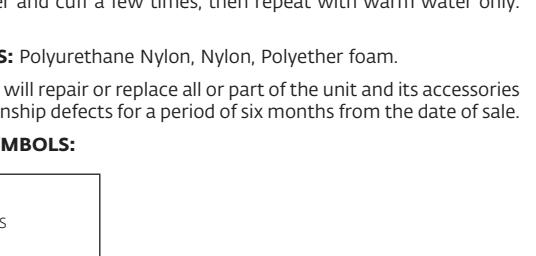FOR SINGLE PATIENT USE ONLY.
NOT MADE WITH NATURAL RUBBER LATEX.

NUR ZUM GEBRAUCH AN EINEM EINZIGEN PATIENTEN.
NO FABRICADO CON LÁTEX DE CAUCHO NATURAL.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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CONTRAINDICATIONS: La crioterapia no debe usarse en personas con Raynaud's disease, cold hypersensitivity, decreased skin sensitivity, or other vasospastic disease, cold hypersensitivity, decreased skin sensitivity, compromised local circulation, diabetics, sickle-cell anemia, arteriosclerosis, cryoglobulinemia, hypercoagulable clotting disorders or other peripheral vascular diseases causing ischemia or poor local circulation, local tissue infection, cold allergy, cold hypersensitivity, previous cold injury and paroxysmal cold haemoglobinuria.

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2. **APLICAR EL VACÍO CUFF -** Posicione el cuff so that the quick-disconnect is pointing up, so that it conforms to a slightly bent knee or elbow and secure straps (Fig. 1). Tighten straps to a comfortable compression.

3. **FILL CUFF -** Connect blue tube to the cuff, listen for a click/snap. When using the Gravity Cryo/Cuff (non-motorized), open air vent on cooler lid and raise cooler no more than 15 inches (38 cm) above the cuff (Fig. 2).

4. **REFILL WATER -** Reconnect the tube to the cooler below the cuff, allow the water to mix with the ice and rechill, then raise the cooler if needed.

5. **CLEANING INSTRUCTIONS:** After use, completely drain water from cuff, tube, and cooler. (To drain tube, elevate tube while pressing tip of tube/disconnect.) Store cooler with top off to allow drying. Periodically clean cuff, tube, and cooler using a few ounces of liquid soap added to hot water in cooler. Recipe soap/water mixture between cuff and cooler a few times, then repeat with warm water only.

6. **MATERIAL CONTENTS:** Polyurethane Nylon, Nylon, Polyether foam.

7. **WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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DISPONIBLE SOLAMENTE PARA UN SOLO PACIENTE.
NICHT AUS NATURKAUTSCHUKATE HERGESTELLT.

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