**ENGLISH**

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** To aid in immobilization and protection of range of motion associated with ACL, PCL, LCL, and MCL surgeries and meniscal repairs.

**PERFORMANCE CHARACTERISTICS:** Providing immobilization or controlled movement of a limb or body segment. Providing mild protection to a limb or body segment for acute and prophylactic care.

**CONTRAINdications:** Instable knee, one operative therapy required.

Providing immobilization or controlled movement of a limb or body segment. Providing mild protection to a limb or body segment for acute and prophylactic care.

**CONTRAINdications:** Unstable knees requiring operative therapy, open wounds, cellulitis or lacerations that will be occluded by the brace. Do not use if you are allergic to any of the materials contained within this product.

**WARNINGS AND CAUTIONS:** If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your medical professional immediately.

**LOSS OF CIRCULATION:** Patient discomfort and patient re-injury are potential effects caused by device failure. ROM settings should not be changed without supervision of a medical professional. This product must be prescribed and fitted by a healthcare professional. The frequency, duration of use and directions for use should be determined by your healthcare professional. Please be advised that DJO, LLC does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. The support should be snug but not impair circulation. Rigid knee braces are not recommended to be worn over clothing. Do not use over open wounds. Do not use this device if it is damaged and/or the packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to use of this device.

**APPLICATION INFORMATION:**

- Unfasten strap ends near black buckles and open brace flat by pulling both uprights apart.
- Place leg on top of flat brace. Shorter upright is applied to the thigh. Adjust brace until each upright is aligned to centerline of the medial and lateral sides of the leg and the center of the hinge is aligned with the center of the knee joint.
- Telescopics sliders by depressing the slider button until the desired length is reached. The top and bottom sliders should be adjusted first, followed by the sliders closest to the knee (note that inner sliders cannot be adjusted until the outer sliders are moved to provide sufficient clearance). Ensure that both bars are equal in length and that each strap slider is adjusted to the same position on each upright respectively.
- Ajuste los largos de las correas:

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**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN A THERAPEUTIC PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.**

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