



PHYSIOstrap™ SPORT

INDICATIONS: Wearing the EPITHELIUMFLEX® PHYSIOstrap™ helps relieving patellar conditions (patellar syndrome, patellar instability, patellar tendonitis, etc.). In the event of a knee injury with residual effects (ligament weakness, osteoarthritis, etc.), it helps to stabilise the joint when resuming sports activities.

CONTRAINDICATIONS: To ensure blood circulation is not impeded, do not use in the event of arteritis, severe varicose conditions or residual effects of phlebitis or oedema (lymphatic/venous) of the lower leg and/or thigh. Do not use with compression hosiery. Do not use on an open wound.

PRECAUTIONS FOR USE:

- To avoid any risks of irritation generated by the silicone elastic bands, the PHYSIOstrap™ should not be worn in static positions, particularly with the wearer lying down (sleep) or sitting (including bus or car journeys, etc.). The PHYSIOstrap™ should be put on immediately before activity, to be worn during movement, during sports activities.
- If you use calf compression sleeves (such as Booster), place the lower elastic band of the knee brace over the sleeve.
- Place the PHYSIOstrap™ on the thigh without stretching.

HOLD:

- Do not apply moisturising products before using the PHYSIOstrap™ to ensure optimum hold.
- If you encounter any hold problems in extreme conditions (sports with sudden changes of speed, running on steep slopes or over long distances, high temperatures), we recommend that you wear a thigh sleeve over the PHYSIOstrap™.


IMPORTANT! To guarantee good hold, the product must be washed before each use. At the very least, wash the silicone elastic bands with soap and water (without rubbing) to remove any dirt.

WASHING INSTRUCTIONS: Machine wash at 30°C or hand wash (without rubbing), leave to dry naturally. Do not iron.

COMPOSITION: 70% polyamide/elastane fabric, 30% silicone.

HOW TO FIT

- A) Unfold the elastic bands outwards.
- B) Fold the PHYSIOstrap™ back on itself to slip it on and position the kneecap in the centre of the "support ring".
- C) Unfold the elastic bands **without stretching the PHYSIOstrap™ on your leg.**
- D) Adjust your PHYSIOstrap™ in the standing position **and ensure that the support rings are in a flat position.**

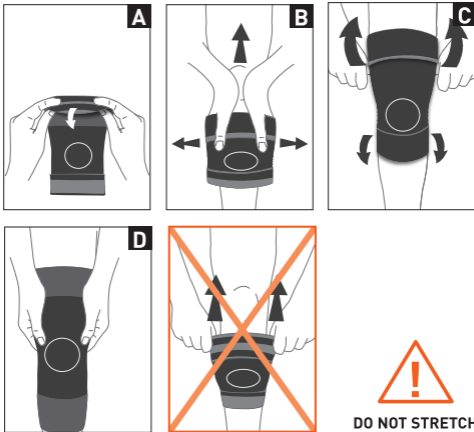

The biocompatibility of the materials has been assessed according to ISO standard is 10993-1 relating to products that come into contact with the skin. If an incident occurs, stop using the product immediately and inform MILLET Innovation.

If you experience pain, oedema, discomfort or any other abnormal reaction, stop using the product and consult a doctor. MILLET Innovation cannot be held liable in cases where the manufacturer's recommendations have not been followed during use. French is the reference language. CE initial marking: 2014.



Machine washable at 30°C.



Do not iron or steam.



Do not dry clean.



Do not bleach.



Do not tumble-dry.



See packaging.



MILLET Innovation
ZA Champgrand 26270 Loriol/Drôme - FRANCE
Product of MILLET Innovation research

