ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: The OA REACTION WEB knee brace is designed to help reduce osteoarthritis knee pain by off-loading the affected joint compartment. Indications for use are mild to moderate unicompartmental osteoarthritis of the knee and patella, and post-operative rehabilitation.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

1. Pull on sleeve until comfortable and patella is centered in the hour glass shaped opening.
NOTE: Wearing sleeve is optional.

2. Lay web over top of knee joint, centering web by aligning opening with patella (knee cap).
NOTE: Outside of web has the DonJoy logo, inside of web has dimples.

3. Ensure off-loading hinge (with silicone pad) is placed at the center of the knee joint.

4. Feed loose straps pad through openings on the opposite side of the off-loading hinge on calf. Then secure strap to black loop side of back pad.

- 4b. Repeat on thigh.
NOTE: Grey side of back pad should be placed on skin and loop black side should be away from leg.

- 5a. Once all straps are affixed, to ensure proper tightness lift and pull both of lower (calf) straps until snug.

- 5b. Repeat for top (thigh) straps – brace should be snug and comfortable, do not over tighten.

OFF-LOADING HINGE ADJUSTMENT: Place off-loading hinge on the edge of a table. Place the thigh upright on the edge of the table about 1" above the hinge and press downward on the calf portion of the upright to create a bend in the upright. Repeat on the lower (calf) portion if necessary.

NOTE: Ensure patient has comfortable amount of pressure from the off-loading hinge.

CLEANING INSTRUCTIONS: The web, plastic parts and softgoods can be hand washed with mild soap in lukewarm water. Rinse both thoroughly. AIR DRY separately.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY

RX ONLY

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SOLO CON RECETA MÉDICA

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN A THERAPEUTIC PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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